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EN | Wound Drainage System / replacement bottle (high vacuum and middle vacuum).
Sterile ETO/Gas, disposable product, use only in undamaged packaging inside and
outside sterile, pyrogen free and only with "Max" vacuum indicator. When reusing
infection risk! Application by qualified personnel only.
Safety instructions on: www.oriplast.de

USE OF THE BOTTLE (THEATRE SYSTEM):

1. Open the peel pack under sterile conditions.
2. Remove the bottle from the secondary packaging sterile.
3. Do not use the bottle when the measuring sensors are pointing to "zero".
4. Cut the cut down connector to the required French gauge (FG).
5. Connect the drain the cut down connector or tighten the screw connector around the drain.
6. Open the closed slide clamp on the bottle.
7. Suction process begins by itself.

ATTENTION: The bottle is vacuum-free when the the measuring sensors are
pointing to "zero". The secretion bottle must then be replaced with a new evacuated
bottle (exchange bottle).

CONNECTION OF THE REPLACEMENT BOTTLE (WARD EXCHANGE SYSTEM):

- a) Close the slide clamp on the connection tube (if available).
- b) Close the slide clamp on the bottle and remove the connecting tube from the red rubber piece or the QL screw connection.
- c) Connect the new bottle to the connecting tube.
- d) Open the slide clamp on the connection tube (if available).
- e) Open the slide clamp at the bottle.
- f) Suction begins by itself.

ATTENTION: No household waste. The used bottles should be disposed of properly
after use.

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