THORAFLEX

**THORAFLEX • THORAFLEX I • THORAFLEX III • THORAFLEX VI
THORAFLEX VIII • THORAFLEX XXXV • THORAFLEX XXXIV**

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EN | Sterile ETO/Gas – For single use – sterile and pyrogen-free only in undamaged packaging inside and outside. When reusing infection risk! Application only by qualified personnel only. Safety instructions on: www.oriplast.de

Instruction manual

One-bottle systems Standard:

THORAFLEX • THORAFLEX I • THORAFLEX III • THORAFLEX VI

REF VK00053, VK00054, VK00055, VK00056, VK00295

1. Fill the bottle with 250 ml sterile water.
2. Screw the tube system tightly. The connection must be tight.
For the safe function of the water lock (backflow protection), screw the riser tube. Fix the riser tube up to the mark (approx. 3 cm from the lower end of the tube) below the water surface.
3. Connect the tube with the connector to the drainage.
If suction is desired, connect the 2nd tube to the vacuum source.

Two-bottle systems Standard:

THORAFLEX VIII • THORAFLEX XXXV • THORAFLEX XXXIV

REF VK00061, VK00065, VK00296

1. Fill one bottle with 250 ml sterile water.
2. Screw the tubing system tightly onto both bottles. Screw cap with long riser tube onto the bottle filled with water. The connection must be tight.
For the safe function of the water lock (backflow protection), push the riser tube. Fix the riser tube up to the mark (approx. 3 cm from the lower end of the tube) below the water surface.
3. Connect the tube with the connector to the drainage.
If suction is desired, connect the 2nd tube to the vacuum source.

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