

THORAFLEX

THORAFLEX XXXII

DE | **THORAFLEX** Zweiflaschensystem

EN | **THORAFLEX** Two-bottle system

DE | **Steril ETO/Gas – Einmalprodukt – steril, pyrogenfrei nur in unbeschädigter Verpackung innen und außen. Bei Wiederverwendung Infektionsgefahr! Anwendung nur durch geschultes Personal. Sicherheitshinweise auf: www.oriplast.de**

Gebrauchsanweisung

THORAFLEX XXXII

 VK00064

1. Flasche 1 (Sogregulierung) mit 1000 ml sterilem Wasser befüllen.
2. Flasche 2 (Sekret auffang / Wasserschluss) mit 250 bis 300 ml sterilem Wasser befüllen.
3. Schlauchsystem fest aufschrauben. Verbindung muss dicht sein. Das Steigrohr zur sicheren Funktion des Wasserschlusses (Rückflusssicherung) bis zur Markierung (ca. 3 cm vom unteren Rohrende) unter der Wasseroberfläche fixieren.
4. Schlauch mit Konnektor an Drainage anschließen. Den 2. Schlauch (ohne Konnektor) an die Vakuum-Quelle anschließen.
5. Mittels Tauchstab an Flasche 1 den gewünschten Sog einstellen; Graduierung an der Oberkante des grünen Stützens ablesen.

System generell zur korrekten Funktion unterhalb des Patientenniveaus anbringen. System aufrecht stellen und transportieren.

EN | **Sterile ETO/Gas – For single use – sterile and pyrogen-free only in undamaged packaging inside and outside. When reusing infection risk! Application only by qualified personnel only. Safety instructions on: www.oriplast.de**

Instruction manual

THORAFLEX XXXII

 VK00064

1. Fill into bottle 1 (suction control) 1000 ml of sterile water.
2. Fill into bottle 2 (secretion collection / water seal) 250 – 300 ml of sterile water.
3. Screw the tubing system tightly onto the bottle and fix pipe into the water in the way, the mark reaches water-surface.
4. Connect the tube (cone-connector) with the drainage. Connect the second tube (without connector) with vacuum.
5. Adjust the wished suction force by moving the graduated pipe of the bottle 1; the suction force is indicated on graduation at top – line of green carbine.
6. Place the complete system in a secure holder.

To assure the systems function, systems should be placed below patient-level! Place the system upright and transport.

 0297

STERILE EO



 **ORIPLAST®**

ORIPLAST GmbH
Hermannstrasse 150
66538 Neunkirchen
Germany

Phone +49 (0) 6821 / 80 47
Fax +49 (0) 6821 / 82 61
Mail info@oriplast.de
www.oriplast.de