PLEURAL PUNCTURE KIT

EN | INSTRUCTIONS FOR USE

read all instructions carefully before use: failure to follow specific instructions and precautions may result in complications of the procedure. **ASEPTIC TECHNIQUE IS REQUIRED.**

EQUIPMENT DESCRIPTION

The pleural puncture set consists of a three-way stopcock, a puncture cannula, a transfer tube with lock attachment, a disposable syringe 50 ml and a secretion bag 1.5 l.

USE

Handling by professional personnel only.

Puncture must be carried out under aseptic conditions. Connect the cannula and tip to the three-way stopcock or extension tube. Shut off the cannula with the three-way stopcock in the appropriate position. Perform the puncture and advance the cannula into the pleural space. Draw secretion into the syringe and transfer it to the bag. Withdraw the cannula from the pleural space.

Indications:

- Removal of secretions from the pleural space
- Pneumothorax
- Irrigation of the pleural space
- sampling
- drug installation

CONTRAINDICATIONS

There are no absolute contraindications.

Relative contraindications:

- Bleeding disorder or anticoagulation
- Unsafe location of the fluid
- Minimal fluid volume
- Altered anatomy of the chest wall
- Pulmonary disease severe enough to cause life-threatening complications
- Uncontrolled cough

WARNINGS AND COMMON MISTAKES

Be careful to insert the pleural puncture needle just above the top of the rib and not below it to avoid the intercostal blood vessels and nerves at the bottom of each rib.

Do not use if inner packaging is damaged.

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